



InteleViewer: Log in and set up a bookmark

Step 1


Open InteleViewer and then click on 'Edit Bookmarks' icon.

Note: You will need to add a new bookmark for HIS when you open Inteleviewer for the first time. This only needs to be done once.



Step 2

A new window will open. Click the 'Add' button that looks like a green plus sign.

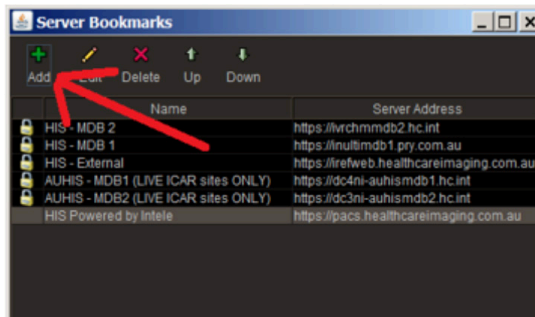


Step 3

In the Server URL field, enter the PACS server address:
<https://pacs.healthcareimaging.com.au>


Tick 'Protect Bookmark' and 'Use Custom Server Name'.
In the 'Name' Field, type 'HIS Powered by Intele'.

Click Ok.

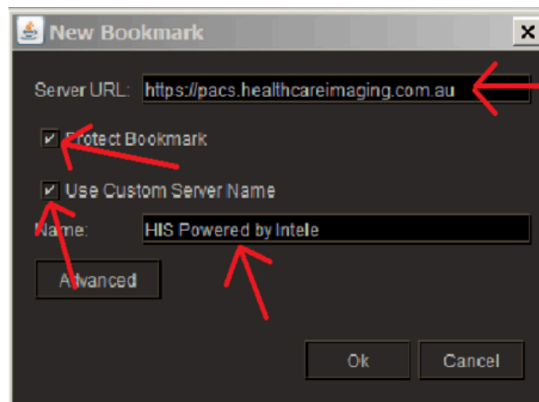

Step 4

Enter your username and password and click on the grey arrow to log in, or press Enter on your keyboard.

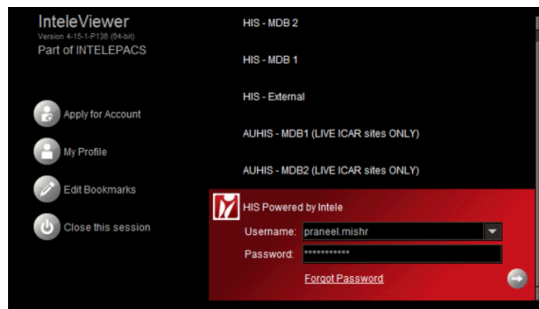

Step 5

Inteleviewer will open in the search screen as shown.

